Echelon

Instructions for Use

EC22L1S-EC30R8S
EC22L1SD-EC30R8SD

EN	Instructions for Use	1
FR	Instructions D'Utilisation	15
DE	Gebrauchsanweisung	29
IT	Istruzioni per L'Uso	43
ES	Instrucciones de Uso	57

1 Description and Purpose

Application

These instructions are for the practitioner.

The Echelon foot is to be used exclusively as part of a lower limb prosthesis.

This device provides limited self-alignment of the prosthesis on varied terrain and following footwear changes. It is intended to improve on postural sway and symmetry while easing abnormal pressures at the socket interface. A moderate-energy-return foot with multi-axial ankle movement. Independent heel and toe springs provides some axial deflection. The split toe provides good ground compliance.

This device is recommended for users that have the potential to achieve Activity Level 3 who may benefit from enhanced stability and an increase in confidence on uneven surfaces.

Of course there are exceptions and in our recommendation we want to allow for unique, individual circumstances. There may also be a number of users in the Activity Levels 2 and 4* who would benefit from the enhanced stability offered by Echelon, but this decision should be made with sound and thorough justification.

*Maximum user weight 100 kg and always use one higher spring rate category than shown in the Spring Set Selection table.

⚠ To minimise the risk of slipping and tripping, appropriate footwear that fits securely onto the footshell must be used at all times.

⚠ Be aware of finger trap hazard at all times.

Contra-indications

This device may not be suitable for Activity Level 1 individuals or for competitive sports events, as these types of users will be better served by a specially-designed prosthesis optimized for their needs.

Intended for a single user.

Ensure that the user has understood all instructions for use, drawing particular attention to the section regarding maintenance.

Spring Set Selection

		User Weight								
Activity		44-52	53-59	60-68	69-77	78-88	89-100	101-116	117-125	kg
		(100-115)	(116-130)	(131-150)	(151-170)	(171-195)	(196-220)	(221-255)	(256-275)	(lb)
3		1	2	3	4	5	6	7	8	Foot Spring set

Activity Level 3

Has the ability or potential for ambulation with variable cadence. Typical of the community ambulator who has the ability to traverse most environmental barriers and may have vocational, therapeutic, or exercise activity that demands prosthetic utilization beyond simple locomotion.

Note:

If in doubt choosing between two categories, choose the higher rate spring set.


Foot Spring set recommendations shown are for transtibial users.

For transfemoral users we suggest selecting a spring set one category lower, refer to fitting advice Section 7 to ensure satisfactory function and range of movement

2 Construction

Principal parts:

- Hydraulic Body Assembly including pyramid (aluminum/St. Stl./titanium)
- Carrier Assembly (aluminum/St. Stl.)
- Heel & Toe Springs (e-carbon)
- Spring Attachment Screws (titanium/St. Stl.)
- Glide Sock (UHM PE)
- Foot Shell (PU)

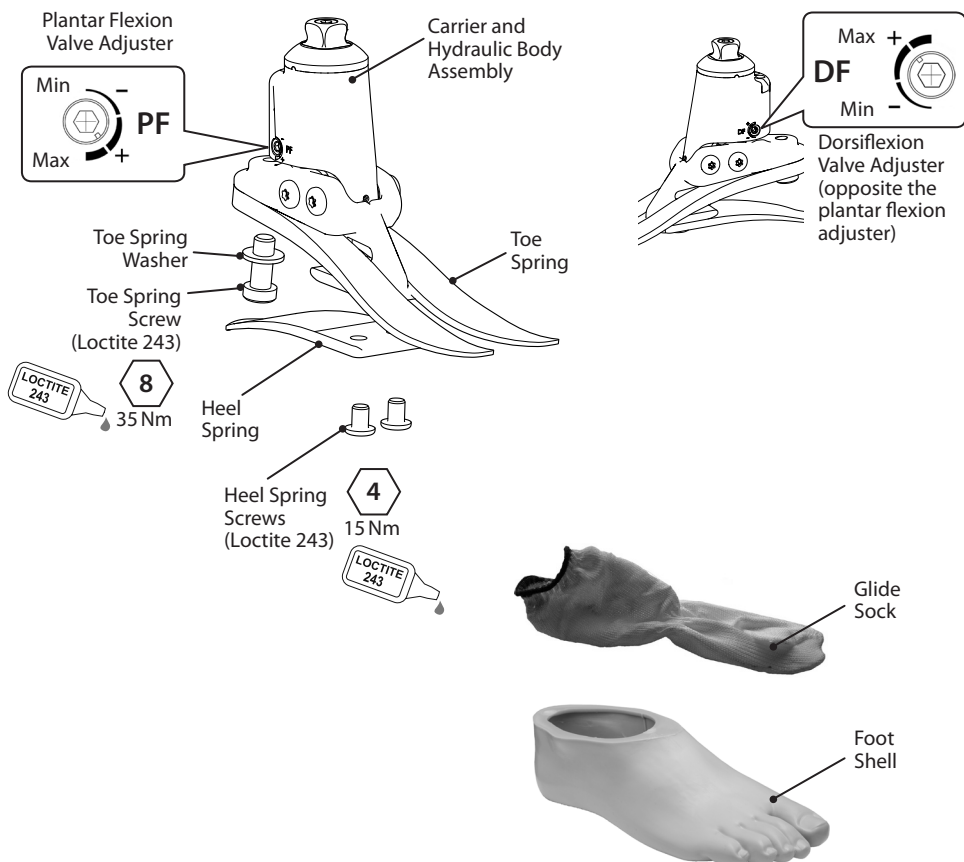3 Function

Echelon comprises a hydraulic body assembly containing adjustable hydraulic valves. The valves can be independently adjusted to increase and reduce hydraulic resistance of plantar and dorsiflexion. The hydraulic body assembly is connected to a carrier assembly via two pivot pins. Heel and toe springs are attached to the carrier assembly using titanium and stainless steel screws. The foot is wrapped in a UHM PE sock which is in turn surrounded by a PU foot shell.

4 Maintenance

Maintenance must be carried out by competent personnel.

It is recommended that the following maintenance is carried out annually:

- Remove the foot shell and glide sock, check for damage or wear and replace if necessary.
- Check all screws for tightness, clean and reassemble as necessary.
- Visually check the heel and toe springs for signs of delamination or wear and replace if necessary. Some surface damage may occur after a period of use, this does not affect the function or strength of the foot.

The user should be advised:

Any changes in performance of this device must be reported to the practitioner.

Changes in performance may include:

- Increase in ankle stiffness
- Reduced ankle support (free movement)
- Any unusual noise

The practitioner must also be informed of any changes in body weight and/or activity level.

The user should be advised that a regular visual check of the foot is recommended, signs of wear that may affect function should be reported to their service provider (e.g. significant wear or excessive discoloration from long term exposure to UV).

Cleaning

Use a damp cloth and mild soap to clean outside surfaces, do not use aggressive cleansers.

5 Limitations on Use

Intended life

A local risk assessment should be carried out based upon activity and usage.

Lifting loads

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

Environment

This device is waterproof to a maximum depth of 1 meter.

Thoroughly rinse with fresh water after use in abrasive environments such as those that may contain sand or grit, for example, to prevent wear or damage to moving parts.

Thoroughly rinse with fresh water after use in salt or chlorinated water.

Foot products must be adequately finished to prevent water ingress into the foot shell where possible. If water enters the foot shell, the limb should be inverted and dried before further use.

Exclusively for use between -15 °C and 50 °C (5 °F and 122 °F).


Suitable for submersion We recommend using only Blatchford products with this device.

6 Bench Alignment

6.1 Static alignment


Align transfemoral devices according to fitting instructions supplied with the knee.

Keep the build line between pivots as shown, using shift and/or tilt devices as necessary.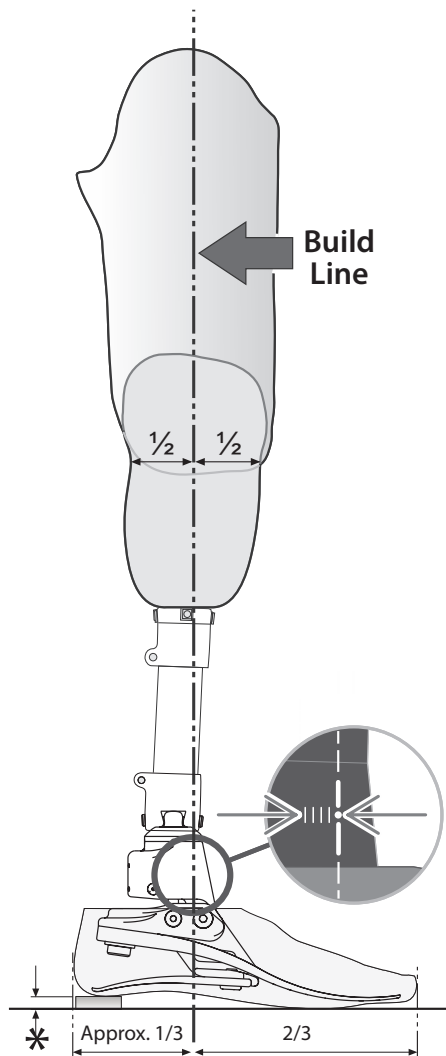

Tilt setting


Align limb to achieve range of motion shown.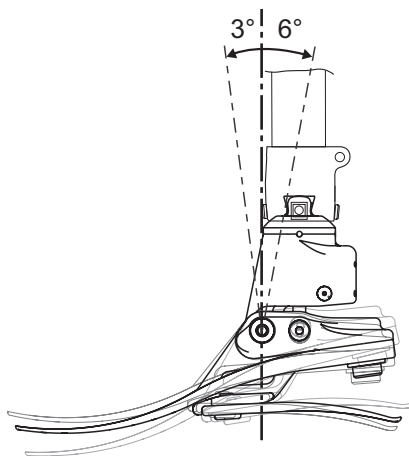

* Allow for users own footwear

6.2 Biomimetic alignment


The aim of alignment is to achieve a “balance point” while standing and set the hydraulically damped range of motion. The aim of damping adjustment is to fine tune the ankle-foot roll-over stiffness characteristics until a comfortable gait is achieved. Due to the increased range of motion provided by the ankle the user may experience the need for more voluntary control and initially find the ankle disconcerting during setup. This should quickly pass upon completion of satisfactory setup.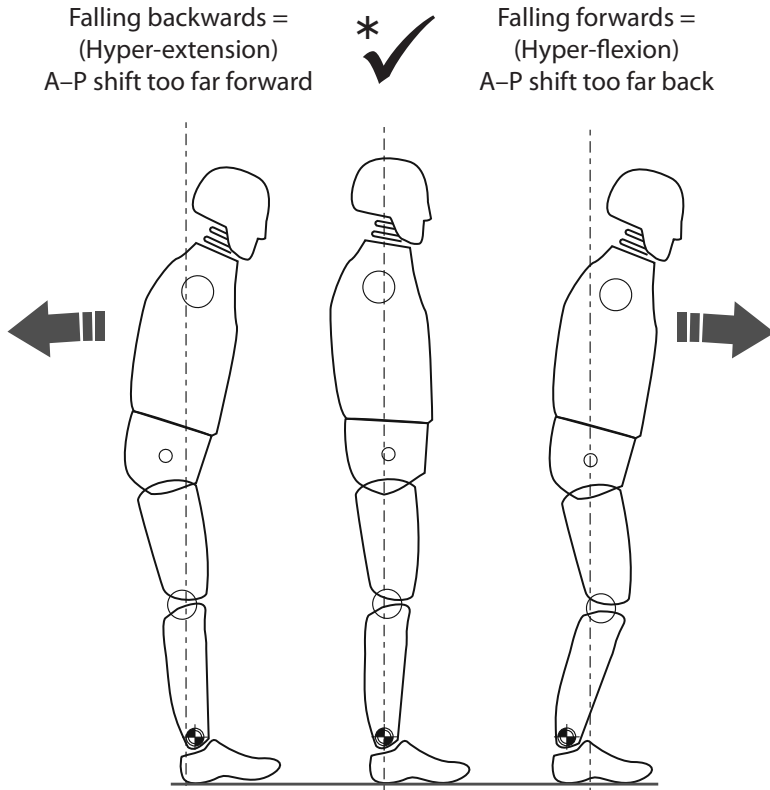

* Ensure that the user is relaxed and not resting on the dorsiflexion limit.

6.3 Biomimetic adjustment

NB: Carry out static alignment while ensuring the user has some means of support such as parallel bars. This is standing alignment only.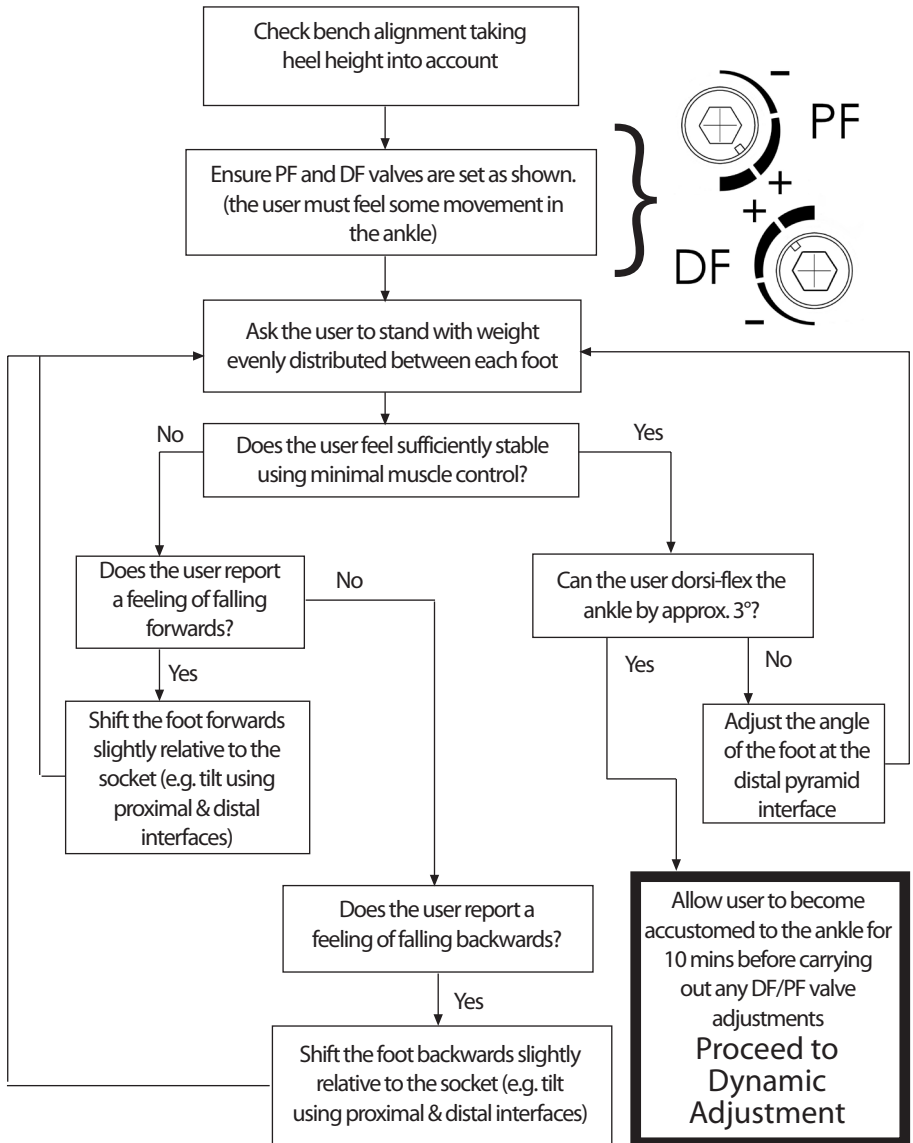


Use shift for static alignment and standing.

The device should encourage some degree of self adjustment to achieve a sense of balance for the user during standing.

6.4 Dynamic adjustment

Adjustment of the hydraulic valves.

The user should experience the ankle moving with the body through the gait cycle. There should be no effort exerted by the user to overcome the hydraulic resistance of the ankle.

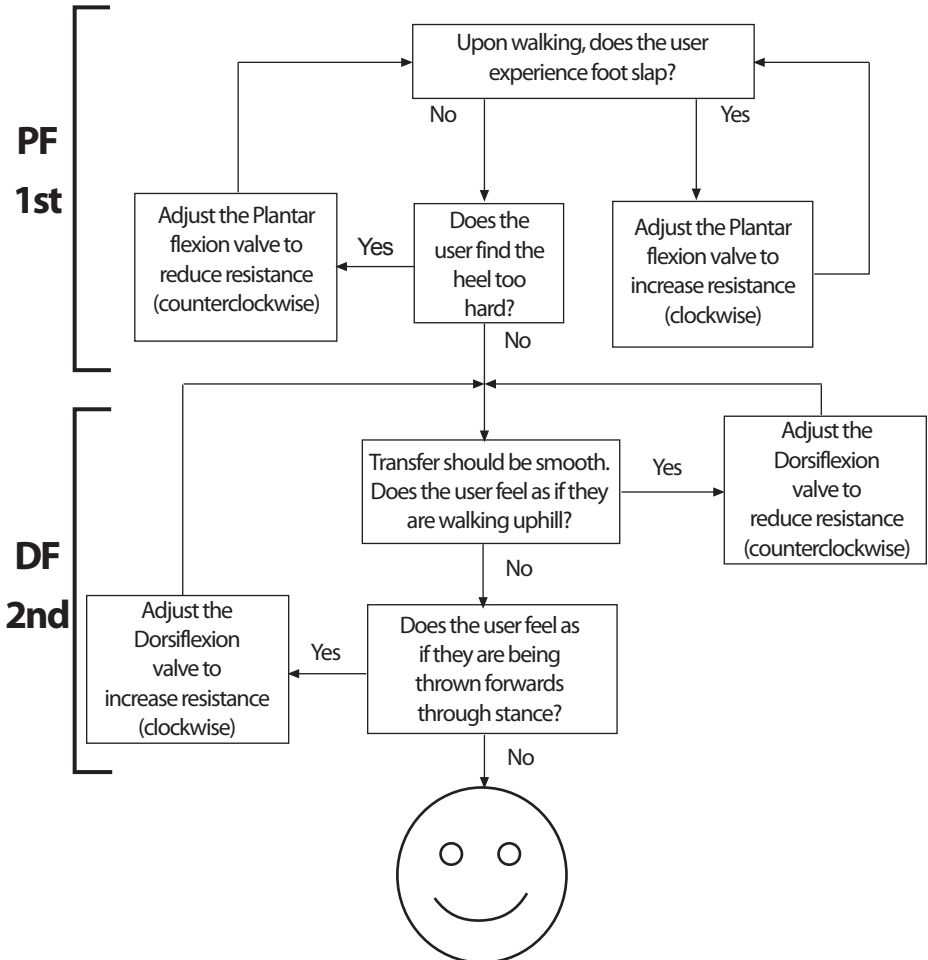Guidance:

Following dynamic adjustment, trial the foot/ankle on ramps and stairs. Ensure the user is comfortable with the kind of terrain he/she may normally be expected to encounter. If the user reports any issues with comfort, usability or range of movement of the ankle, adjust accordingly.

7 Fitting Advice

The correct alignment (A–P position), range of motion (distribution of plantar to dorsiflexion) and adjustment of the hydraulic settings are critical in achieving a smooth roll over and correct slope adaptation (see 6.3).

The springs for the Echelon foot will be supplied assembled with heel and toe springs of the same category. If after following the instructions below you still have problems with the function please contact the sales team in your area for advice.

Any of the following:

- Incorrect spring selection
- Incorrect A–P shift alignment
- Incorrect distribution of plantar and dorsiflexion range will have a negative effect on function and stability

	Symptoms	Remedy
1.	<p>Sinking at heel strike</p> <p>Difficulty in achieving a smooth progression to mid stance</p> <p>User feels they are walking up hill or forefoot feels excessively long</p>	<ol style="list-style-type: none"> 1. Increase plantar flexion resistance 2. Check A–P shift alignment; ensure foot is not too anteriorly positioned 3. Check distribution of plantar and dorsiflexion movement; ensure that the plantar flexion range is not excessive 4. Check spring category is not too soft, if so fit a higher rate spring
2.	<p>Progression from heel strike to mid stance is too rapid</p> <p>Difficulty in controlling the energy return from the foot at the heel strike (reduced knee stability)</p> <p>User feels heel is too hard, fore foot is too short</p>	<ol style="list-style-type: none"> 1. Reduce plantar flexion resistance 2. Check A–P shift alignment; ensure foot is not too posteriorly positioned 3. Check distribution of plantar and dorsiflexion movement; ensure that there is adequate plantar flexion range 4. Check the spring category is not too high for the weight and activity of the user, if so fit lower rate spring
3.	<p>Heel contact and progression feel OK but:</p> <p>Forefoot feels too soft</p> <p>Forefoot feels too short</p> <p>User feels they are walking down hill, possibly with reduced knee stability</p> <p>Lack of energy return</p>	<ol style="list-style-type: none"> 1. Increase dorsiflexion resistance 2. Check A–P shift alignment; ensure foot is not too posteriorly positioned 3. Check distribution of plantar and dorsiflexion movement; ensure that there is not excessive dorsiflexion range 4. Check the spring category is not too soft for the weight and activity of the user, if so fit higher rate spring


	Symptoms	Remedy
4.	Forefoot feels too rigid Forefoot feels too long Feels like walking up hill	<ol style="list-style-type: none"> 1. Reduce dorsiflexion resistance 2. Check A–P shift alignment; ensure foot is not too anteriorly positioned 3. Check distribution of plantar and dorsiflexion movement; ensure that there is sufficient dorsiflexion range 4. Check the spring category is not too rigid for the weight and activity of the user, if so fit lower rate spring

8 Assembly Instructions

Be aware of finger trap hazard at all times.


Foot shell removal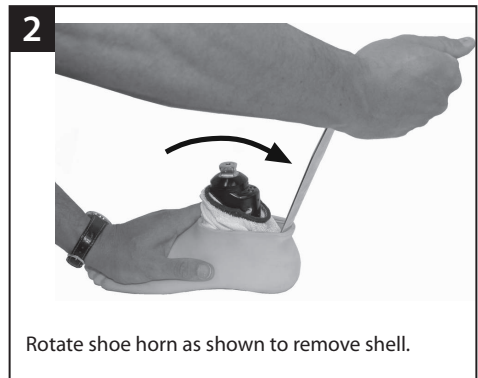

8 Assembly Instructions *(continued)*

Spring replacement

3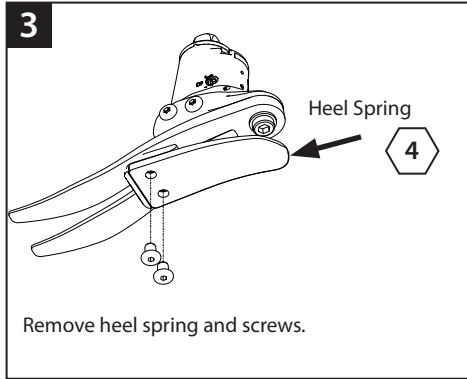


Heel Spring

4

Remove heel spring and screws.

4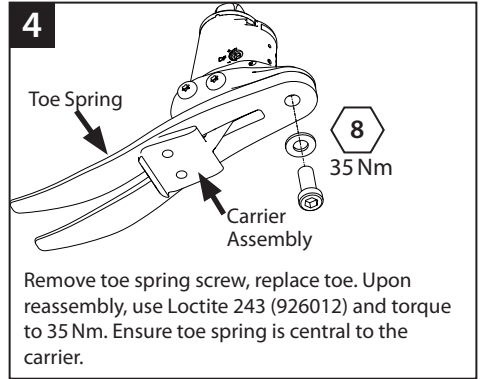


Toe Spring

Carrier Assembly

8
35 Nm

Remove toe spring screw, replace toe. Upon reassembly, use Loctite 243 (926012) and torque to 35 Nm. Ensure toe spring is central to the carrier.


5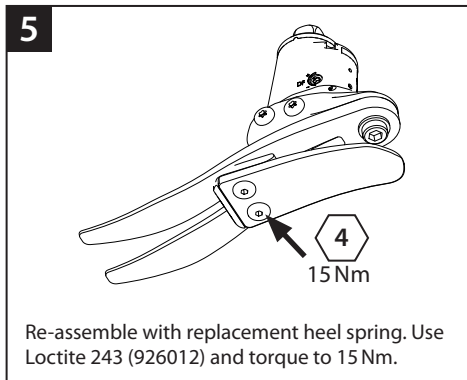

4
15 Nm

Re-assemble with replacement heel spring. Use Loctite 243 (926012) and torque to 15 Nm.

6

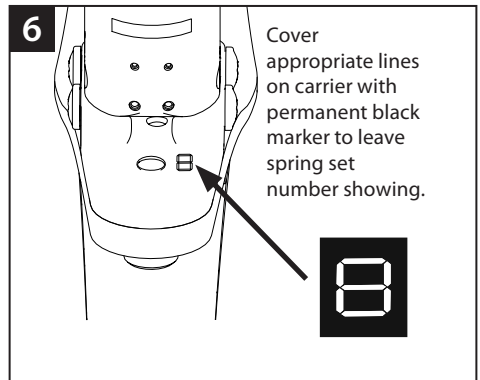Cover appropriate lines on carrier with permanent black marker to leave spring set number showing.

8

7

If a foam cosmesis is to be fitted, roughen top surface of foot shell to provide ideal bonding surface.

8



928017

Lubricate toe and heel if required. (Foot shell is pre-lubricated).

Fit sock as shown.

8 Assembly Instructions *(continued)*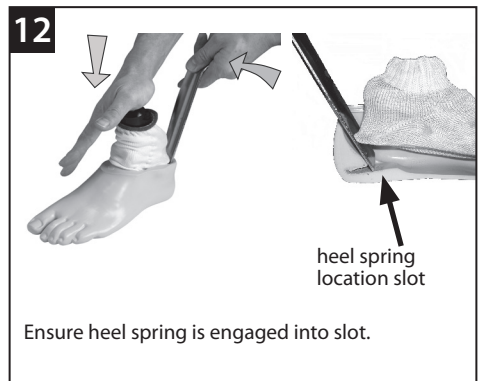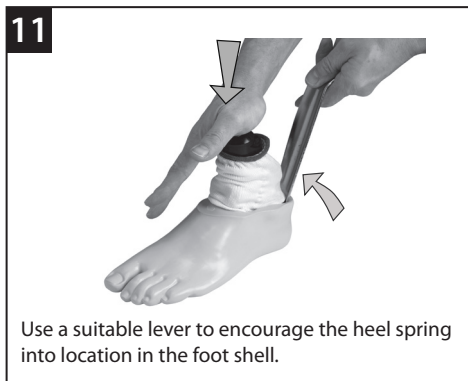


If a cosmetic finish is required please contact a member of the Blatchford Sales Team.

9 Technical Data

Operating and Storage Temperature Range:	-15 °C to 50 °C (5 °F to 122 °F)
Component Weight (<i>Size 26N</i>)	900 g (2 lb)
Activity Level:	3
Maximum User Weight:	125 kg (275 lb)
Proximal Alignment Attachment:	Male Pyramid (Blatchford)
Range of Hydraulic Ankle Motion (excludes additional range of motion provided by heel and toe springs)	6 degrees plantar flexion to 3 degrees dorsiflexion
Build Height: (See diagram below)	(<i>sizes 22–24</i>) 115 mm (<i>sizes 25–26</i>) 120 mm (<i>sizes 27–30</i>) 125 mm
Heel Height	10 mm

Fitting length

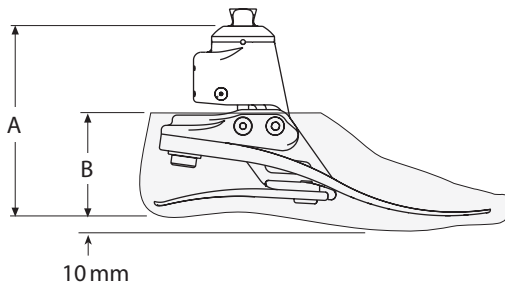Size	A
22–24	115 mm
25–26	120 mm
27–30	125 mm

Size	B
22–26	65 mm
27–28	70 mm
29–30	75 mm

10 Ordering Information

Order Example

EC	25	L	N	3	S
	Size	Side (L/R)	Width* (N/W)	Spring Set Category	Sandal Toe

Available from size 22 to size 30:
EC22L1S to EC30R8S
EC22L1SD to EC30R8SD

*Sizes 25-27 only. For all other sizes, omit the Width field.

(add 'D' for a dark tone foot shell)

e.g. EC25LN3S, EC22R4S, EC27RW4SD

Spring Kits				
Rate	Foot sizes			
	Small (S)	Medium (M)	Large (L)	Extra Large (XL)
	22-24	25-26	27-28	29-30
Set 1	539801S	539810S	539819S	539828S
Set 2	539802S	539811S	539820S	539829S
Set 3	539803S	539812S	539821S	539830S
Set 4	539804S	539813S	539822S	539831S
Set 5	539805S	539814S	539823S	539832S
Set 6	539806S	539815S	539824S	539833S
Set 7	539807S	539816S	539825S	539834S
Set 8	539808S	539817S	539826S	539835S

Foot Shell (for dark add 'D')			Glide Sock	
Size/Side	Narrow	Wide		
22L	539038S	-	531011	
22R	539039S	-		
23L	539040S	-		
23R	539041S	-		
24L	539042S	-		
24R	539043S	-		
25L	539044SN	539044SW		
25R	539045SN	539045SW		
26L	539046SN	539046SW		
26R	539047SN	539047SW		
27L	539048SN	539048SW		532811
27R	539049SN	539049SW		
28L	-	539050S		
28R	-	539051S		
29L	-	539052S		
29R	-	539053S		
30L	-	539054S		
30R	-	539055S		

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

CE Conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty

This device is warranted for 36 months - foot shell 12 months - glide sock 3 months. The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions. See Blatchford website for the current full warranty statement.

Environmental Aspects

Where possible the components should be recycled in accordance with local waste handling regulations.

Manufacturer's Registered Address

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Blatchford SAS
Endolite India Ltd